Bioinformatie Technology

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
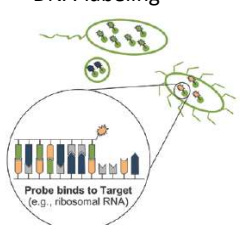
DNA-labeling

Probe binds to Target (e.g., ribosomal RNA)

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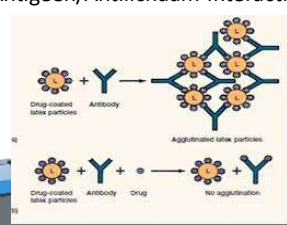Drug-coated latex particles + Antibody → Agglutinated latex particles

Drug-coated latex particles + Antibody + Drug → No agglutination

Bacteriën
Mycoplasma
Gisten
Schimmels
Protozoa


Antilichamen
Antigenen
Virussen
Hormonen
Medicijnen

Copyright Biotrack 2017

ISO16140 validatie

Parameter	Value
Data transformation, description and rationale	Data quality does not give rise to rational transformation
Coefficient of Variation	30%
Bias	0 Events/milliliter
Slope	0.883
Linearity as expressed by the linear correlation coefficient	0.945
Background noise	0 Events/sample
Detection limit (LOD)	3 Events/sample
Quantification limit (LOQ)	10 Events/sample

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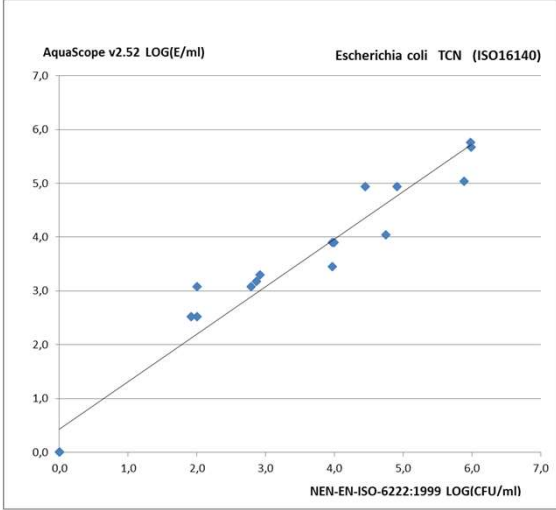Disease	Material	Target(s)**	Sample Preparation	Ready (%)	Under development	Remarks
Lymes disease	tick	Borrelia burgdorferi	Lysis procedure tick	90	Lysis procedure	Increased diagnostic interest because of the increase of this disease
Cystitis	urine	Coliforms	Lysisbuffer	100	ready	
		Pseudomonas				
Sepsis	Blood	Klebsiella				
		Staph aureus	Dilution in phosphate buffered saline	90	Literature survey on cell numbers in clinical and (sub)clinical sepsis patients	Very high prospect application but only is sensitivity AquaScope allows for significant detection
		Enterococci				
	Bloody Pre-culture	Coliforms				
		Staph aureus	Lysis buffer	100	ready	Lot of scientific literature
		Enterococci				
Wound infections	swab	Coliforms				
		Staph aureus	Swab-liquid transfer	90	Sample preparation procedure	
Venereal diseases	swab	Streptococci				
		Candida	Swab-liquid transfer	90	Sample preparation procedure	Especially for home test market, economically relevant, medically less
		Giardia vaginalis		10	Sample preparation procedure & probes development/validation	
		Chlamydia			Sample preparation procedure & probes development/validation	
		Trichomonas			Sample preparation procedure & probes development/validation	
Hygiene & sterility	swabs	MRSA cs	Swab-liquid transfer	100	Validated and medically approved tools for swab-liquid transfer commercially available.	

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Biotrack B.V.
Biotrack Diagnostics, Inc.